

6019 - SURFACTO™ EP-60+

Quick Surface Disinfectant; Alcohol Based



INDICATION

SURFACTO™ EP-60+ is a ready-to-use, aldehyde-free quick disinfectant based on ethanol. It is designed for all surfaces where quick disinfection is required, e.g.: a) patient beds, bedside tables, treatment chairs, vehicles for patient transport, b) alcohol resistant surfaces such as bathtubs, toilet seats and transport baskets, c) alcohol resistant non-invasive medical devices such as thermometers, sphygmomanometers and venous tourniquets.

COMPOSITION

Didecyl dimethyl ammonium chloride (0,05%), benzalkonium chloride (0,05%), ethanol %96 (47%), chlorhexidine digluconate solution 20% (0,5%), isopropanol (13%), pH adjuster (0,01%), corrosion inhibitors (0,20%) and RO water.

MICROBIOLOGICAL PROPERTIES

Bactericide, fungicide, mycobactericide and virucide at 30 seconds.

PACKAGING

1 liter sprayer bottle

6019-1000

INSTRUCTION

Spray to the surface completely. The surfaces must be completely wetted during the required exposure time. Wait for 30 seconds. Clean with a sterile wipe. SURFACTO™ EP-60+ can be used as a wipe disinfectant also.

-  Ready-to-use.
-  Wide material compatibility.
-  Quick efficacy: 30 seconds enough.
-  Quick drying: does not leave trace or greasy film.
-  Free of aldehyde.


medical proximity™
Tel/Fax : +213 (0) 35 33 80 60
Mobile : +213 (0) 560 00 22 70
Email : medproximity.dz@gmail.com
www.medicalproximity.com

ISO 9001:2015
EN ISO 13485:2016




Gül Biyoloji Laboratuvarı

GBL Gül Biyoloji Laboratuvarı Sanayi ve Ticaret Limited Şirketi
Dudullu İMEŞ San. Sit. C Blok 305 Sok. No:16 Ümraniye, İstanbul, Turkey
Phone: +90 216 364 15 00 - Fax: +90 216 314 15 69
Web: www.gbl.com.tr/en - E-mail: export@gbl.com.tr

GMP
ISO 22716:2007

