

## “Air Disinfection Solution”



# 6140 PEROXY™ READY-AIR

PEROXY™ READY-AIR is a surface bio-disinfection product. It is a ready-to-use solution with a base of 7,5% hydrogen peroxide.

### Composition

Hydrogen peroxide 7,5 %, Silver 17 ppm, scent and stabilizing agents.

### Product Specification

Form	Liquid
Color	Colorless
Water Solubility	Soluble
Odor	Pine scent
pH @ 25°C	6,5±1,0

### How to Use

Ready-to-use. Do not dilute. After cleaning the room. Attach the 1L bottle to the diffusion appliance. On the device, set the volume (V) of the room to be treated. As an example: a 20m<sup>2</sup> room with a height of approximately 2,50 m will have a volume of 20 x 2,50m = 50m<sup>3</sup>. The device will have to be set on 50m<sup>3</sup>. After the end of diffusion, respect a dwell time of 30 minutes minimum. During diffusion time and dwell time, leave the room closed and do not enter. The treatment must be conducted with no human presence inside the room. The treatment has to be repeated every day.

### Precautions

Keep away from children, food and animal foods. Causes skin irritation. Wear protective gloves/protective clothing/eye protection/face protection. IF ON SKIN: Wash with plenty of soap and water. If skin irritation occurs: Get medical advice/attention. IF INHALED: Remove person to fresh air and keep comfortable for breathing. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lense if present and easy to do – continue rinsing.

### Storage

Product: in the shade and original packaging, the package should be kept tightly closed and in a well-ventilated environment.

Storage Temperature	Shelf Life	Usage Time After Opening the Cap
15/25 °C	2 years	2 months

### Packaging

1 carton: 5 L jerrycan with 51 mm cap x2

1 pallet: 90 cartons / 180 jerrycans

Order No: 436140035000